

Illegal drug consumption contributes to many accidents, injuries and medical conditions. Screening individuals for drugs of abuse is an important method in identifying those who may cause harm to themselves and to others.

**The Oratect® III Oral Fluid Screen Device** is simple to use for the detection of drugs of abuse in oral fluid



**Drug testing as easy as A,B,C.....**

**1** Remove the blue cap by holding the sides and pulling gently. This will expose the collection pad. **Make sure there is a blue line present in each window area indicating an unused device.**

**2** Open mouth and gently rub the collection pad inside the mouth against cheek in a circular motion several (approximately 15-20) times. Make sure to **keep head level**

**3** Gently **rub** the collection pad against the opposite cheek in circular motion several (approximately 15-20) times.

**4** Gently **rub** the collection pad on top of the tongue several (approximately 15-20) times. **Do not chew, suck, bite or bend the collection pad.**

**5** Rub the collection pad **underneath** the tongue several (approximately 15-20) times.

**6** Place the collection pad underneath the tongue for (approximately 30 seconds to collect saliva. Instruct the donor to hold the device in place with hand. **Sufficient amount of saliva collected is indicated by the flow of the blue lines. Repeat steps 2-6 until blue lines flow.**

**7** Remove from mouth as soon as blue lines flow at both of the testwindows. **Re-cap** the device.

**8** Lay the device on a flat surface and **read results in approximately 5 minutes after removing the device from mouth. Do not read results after 20 minutes.**

**Warnings and Precautions**

The test device should remain in its original sealed pouch until ready for use. Discard the test device if the package is torn. Do not use test device beyond the expiration date indicated on the kit

Handle all oral specimens as potentially infectious. Proper handling and disposal methods should be established.

**Product storage:** Device should be stored at room temperature (15-30 degrees C)



**Alcohol and Drug  
testing specialists**

# Oratect®

**Disposable saliva test  
Rapid screening for drugs anywhere,**



**Saliva sample collection in 1-2 minutes.  
Test accurately for 6 drugs in 5 minutes**

**Drug testing as easy as A,B,C  
Rapid screening for drugs of abuse in oral fluid**

Industry leaders for over 40 years, find out why over 5000 businesses trust our products and expert level support in policy development, legal advice and after sales service

**visit [www.alcosafe.co.za](http://www.alcosafe.co.za) for more information**